

## Rapid Infuser Compatible Solutions

Solution	Description	Compatible?
Anticoagulated Cell-Saved Blood		YES
Processed Anticoagulated Whole Blood		YES
FFP	Fresh Frozen Plasma	YES
RBCs	Red Blood Cells	YES
NS	0.9% NaCl	YES
Albumin 5%		YES
Hydroxyethyl Starch (HES)	Hetastarch in 0.9% Saline	YES
Normosol	Electrolytes in H <sub>2</sub> O	YES
Plasma-Lyte A		YES
Colloids	This is a broad-spectrum term	
Sodium Bicarbonate Solutions		NO
½ NS	0.45% NaCl	NO
3% NS	3% NaCl	NO
Platelets	Should not be diluted, stick to tubing	NO
Cryoprecipitate	Should not be diluted	NO
Calcium containing Solutions	Ca	NO
Lactated Ringer's Solution	K, Na, Cl, Ca, Lactate	NO
Ringer's Solution	K, Na, Cl, Ca, Lactate	NO
Hextend	Hetastarch in Lactated Ringer's	NO
8% Amino Acids		NO
Intralipids 10%		NO
Intralipids 20%		NO
D5W	5% Dextrose in Water	NO
D10W	10% Dextrose in Water	NO
D20W	20% Dextrose in Water	NO
D50W	50% Dextrose in Water	NO
D5 ¼ NS	5% Dextrose 0.2% NaCl	NO
D5 ½ NS	5% Dextrose 0.45% NaCl	NO
D5NS	5% Dextrose 0.9% NaCl	NO
D10NS	10% Dextrose 0.9% NaCl	NO
10% Dextran in 5% Dextrose		NO
10% Dextran 40 in 0.9% NS		NO
5% Alcohol in 5% Dextrose		NO
D5 LR	5% Dextrose in Lactated Ringer's	NO
D10 LR	10% Dextrose in Lactated Ringer's	NO
Glucose		NO